

# Accu-Clear®

## 2 minute pregnancy test

prueba de embarazo de dos minutos


**Over 99% Accurate\***  
from the day of your expected period.

**READY TO TEST?**  
Use Accu-Clear Pregnancy Test for results 5 days sooner than missed period.

**BEFORE YOU BEGIN:**

- Read instructions **COMPLETELY** and have a watch, clock, or timer available.
- Check expiration date on box. Do not use if expired.

**HOW TO USE:**

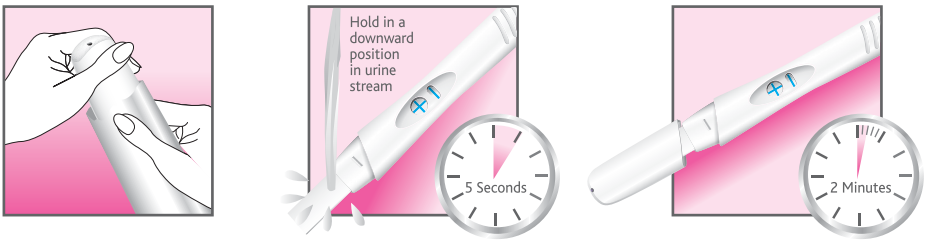

1. Open the protective foil pouch **IMMEDIATELY** before use.
2. Remove the test stick from the foil package. Take off the cap.
3. Hold the test stick by the easy grip handle. **Point the Absorbent Tip downward** in urine stream for **5 seconds ONLY. IMPORTANT: ONLY 5 SECONDS.** Too much or too little urine on Absorbent Tip may cause an invalid result. (See Invalid Result section below.)
4. If you prefer, urinate into a clean, dry cup and dip all of the Absorbent Tip into the cup for **20 seconds.**
5. Replace cap and lay on flat surface with the Result and Control Windows facing up.

**HOW TO READ RESULTS:**

You can read your results after **2 minutes**. A blue line will appear in the Control Window indicating the test has worked. A "-" in the Result Window indicates a negative result. A "+" in the Result Window indicates a positive result.

- **DO NOT** read result after 10 minutes.
- If after 10 minutes **NO** line appears in the Control Window, the result is invalid. Repeat with new Accu-Clear test. (See Question # 7)

**RESULT AREA**

<p><b>Positive Result</b> One blue line in Control Window <b>AND</b> a "+" in Result Window.</p>	<p><b>Negative Result</b> One blue line in Control Window <b>AND</b> a "-" in Result Window.</p>	<p><b>Invalid Result</b> No line appears in Control Window.</p>
--	--	---

The appearance of the "+" symbol in the Result Window may vary, however, the lines that make up the "+" symbol and the line in the Control Window should be of similar width.

ENGLISH

PLEASE READ INSTRUCTIONS CAREFULLY BEFORE USE

# Accu-Clear®

## 2 minute pregnancy test

prueba de embarazo de dos minutos

**Over 99% Accurate\***  
from the day of your expected period.

**Questions & Answers (Q&A)**

- 1. How does Accu-Clear Pregnancy Test work?**  
Accu-Clear Pregnancy Test detects hCG (human Chorionic Gonadotropin), a hormone present in urine once you become pregnant. A "+" (positive) result indicates that hCG has been detected.
- 2. When can I test?**  
Accu-Clear Pregnancy Test can be used as early as 4 days before your expected period which is 5 days sooner than waiting until you miss your period to test. If you test before your expected period and get a "-" (negative) result, there is still a chance that you may be pregnant. See the chart below with clinical results from early testing with Accu-Clear Pregnancy Test.

Number of Days Before	% of women getting a "+" (positive) result with Accu-Clear Pregnancy Test
-4 Days	53%
-3 Days	74%
-2 Days	84%
-1 Day	87%

- 3. Do I have to use first morning urine?**  
Your first morning uring usually contains a higher concentration of hCG. If testing early (before your expected period) it is recommended you use your first morning urine. Otherwise, you can test at any time of day. Please note, drinking large amounts of fluid before testing may dilute the hCG in your urine and cause an inaccurate result.
- 4. I have irregular periods so I am not sure when my next period is due. How do I know I am testing at the right time?**  
With irregular periods you should test according to your longest cycle in recent months. If you are uncertain when your period is due, we recommend testing at least 19 days from the day you think you became pregnant. If the result is negative you should test again in 3 days because you may have tested too early.
- 5. How soon can I read the result?**  
You can read your result after 2 minutes as long as a blue line has appeared in the Control Window. A "+" (positive) result may appear as early as 1 minute. Disregard all results after 10 minutes.
- 6. Are there any factors that can affect the test result?**
  - Always read the manufacturer's instructions for any medication you are taking before the test.

Questions?  
Call toll-free **1-888-698-5031**  
Monday-Friday, 8:30 a.m.-5:00 p.m. Eastern Time.  
Our professional staff is available to assist you.

- Fertility drugs containing hCG (such as Pregnyl® and Profasi®)\*\* can affect the result. Testing with Accu-Clear Pregnancy Test too soon after administration may give a false "+" (positive) result.
- Other fertility therapies (such as Clomid®\*\*\*, painkillers and the contraceptive pill should not affect the result.
- If you have recently stopped taking the contraceptive pill or are using fertility therapies like Clomid®\*\*\*, your periods may be irregular, leading you to test too soon.
- If you have recently been pregnant (even if not carried to full term) you may get a false "+" (positive) result. In addition, ovarian cysts, ectopic pregnancy, menopause and some very rare medical conditions can give misleading results.
- If you do get unexpected results you should discuss them with your doctor.

- 7. What might affect the accuracy of the result?**
  - If testing before your expected period, the test accuracy will be lower. (See chart in Question 2)
  - Not following the instructions correctly, sampling for too little time or too long, not keeping the test with the absorbent tip pointing down and using a test that is out of date can cause misleading results or no result.
  - If you do get unexpected results, you should check with your health care professional.

**8. My test says I'm pregnant, what should I do?**  
You should see your health care professional who can advise you on what steps you should take next.

**9. My test says I'm not pregnant, but I think I am. What should I do?**  
A "-" (negative) result means no hCG has been detected. Wait 3 days and test again using a new Accu-Clear Pregnancy Test. If you still get a "-" (negative) result and your period has not started, you should see your health care professional.

\*Over 99% accurate at detecting pregnancy hormone. However, some pregnant women may not have detectable amounts of pregnancy hormone in the urine on the first day of the missed period or may have miscalculated the first day of their period.

\*\*Pregnyl® is a registered trademark of Oragnon USA, Inc. Profasi® is a registered trademark of Ares Trading S.A.

\*\*\*Clomid® is a registered trademark of Merrell Pharmaceuticals, Inc.

Questions? Call toll-free 1-888-698-5031  
Monday-Friday, 8:30 a.m. -5:00 p.m Eastern Time.  
Our professional staff is available to assist you.

Do not use a test that is past expiration date.  
Do not disassemble device. Contains small parts. Do not eat any component of this device. If accidentally swallowed, seek medical attention.

Accu-Clear Pregnancy Test is for home use. Store at 36°F to 86°F (2°C to 30°C). Do not freeze. For in vitro diagnostic use. Not for internal use. Keep out of reach of children. Do not use if the foil wrapper is damaged.

Distributed by Procter & Gamble, Cincinnati, OH 45202.  
Accu-Clear is a registered trademark of SPD Swiss Precision Diagnostics GmbH.

© 2019 SPD Swiss Precision Diagnostics GmbH. All rights reserved.